

ONCORE Improv Reagent Vials

Reagent Vials for ONCORE Automated Staining Instrument

Control Number: 904-ONC101-081917

Catalog Number: ONC101 JJ, L

Description:

50 Vials, 100 Vials

Intended Use:

ONCORE Improv Reagent Vials are intended for laboratory use only.

Summary & Explanation:

The ONCORE Improv Vials are "user-fillable" vials that are designed for the utilization of alternate primary antibodies not offered within the ONCORE reagent-line on the ONCORE Automated Slide Staining System.

Each ONCORE Improv Vial is good for up to 30 tests and can be filled using a transfer pipet or pipettor.

Known Applications:

Immunohistochemistry and *in situ* hybridization

Supplied As:

ONC101JJ-50 Vials

ONC101L-100 Vials

Storage and Stability:

Empty vials shall be stored at room temperature. Vials containing reagent shall be stored under proper storage condition per reagent label.

Protocol Recommendations:

Use a transfer pipet or pipettor to fill the ONCORE Improv Vials. Do not remove or dislodge the seal, or use such force that the pipet tip enlarges the opening in the seal. If the seal is removed or dislodged, the Improv Vial should be discarded as the seal is designed to prevent excessive evaporation and must be fitted on to the vial at the factory.

Refer to the ONCORE Automated Slide Staining System User Manual (RFID Editor) for instructions on proper use. **Note:** When filling Improv vials; please calculate 258µl volume per slide and 250µl for the vial dead volume based on an 8.0 ml total fill volume due to evaporation and dead volume considerations.

Limitations:

These vials are non-sterile and are designed to only be used on the ONCORE Automated Stainer.

Technical Support:

Contact Biocare's Technical Support at 1-800-799-9499 for questions regarding this product.