

TTF-1 [SPT24]

Concentrated and Prediluted Monoclonal Antibody
901-3126-052623

BIOCARE
M E D I C A L

Available Product Formats				
Format	Catalog Number	Description	Dilution	Diluent
Concentrate	ACI 3126 A, C	0.1, 1.0 mL	1:100	Renoir Red
Predilute	API 3126 AA	6.0 mL	Ready-to-use	N/A
ONCORE	OAI 3126 T60	60 tests	Ready-to-use	N/A
ONCORE Pro	OPAI 3126 T60	60 tests	Ready-to-use	N/A
VALENT	VLTM 3126 G20	20 mL	Ready-to-use	N/A

Intended Use:

For In Vitro Diagnostic Use

TTF-1 [SPT24] is a mouse monoclonal antibody that is intended for laboratory use in the qualitative identification of TTF-1 protein by immunohistochemistry (IHC) in formalin-fixed paraffin-embedded (FFPE) human tissues. The clinical interpretation of any staining or its absence should be complemented by morphological studies using proper controls and should be evaluated within the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

Summary and Explanation:

Thyroid transcription factor-1 (TTF-1) is a 38 kDa member of the NKX2 family of homeodomain transcription factors. TTF-1 is mostly detected in primary lung adenocarcinomas and small cell carcinomas (1). TTF-1 can be very useful in lung cancers when used in a panel with Desmoglein 3, p40 and Napsin A antibodies (2-3).

Commercially available thyroid transcription factor-1 (TTF-1) monoclonal antibodies 8G7G3/1 and SPT24 have been shown to have different sensitivities in lung adenocarcinomas (LADC) and lung squamous cell carcinomas (SqCC) (4-6). A study by Masai, *et al.* demonstrated that SPT24 was much more sensitive than 8G7G3/1 in LADC (72.4% and 65.4% respectively). However, the study demonstrated that SPT24 stained a higher percentage of lung SqCC (16.8% vs. 1%). Higher sensitivity of SPT24 in lung SqCC has also been shown to be heavily influenced by different detection systems (4-5). Higher sensitivity for LADC versus lung SqCC can be achieved with SPT24, compared to 8G7G3/1, while retaining specificity, by the use of a cut-off value and optimal antibody titer. In an in-house study, SPT24 was titrated to achieve negative staining in normal liver (no cytoplasmic staining observed). A cut-off value of $\geq 10\%$ of tumor cells positive for TTF-1 with a staining intensity of $\geq 1+$ was used to identify TTF-1 positive cases. Using this approach, SPT24 was highly sensitive for LADC (53/60, 88%), compared to 8G7G3/1 (38/60, 63%), with equivalent specificity for both clones versus lung SqCC (2/137, 1.5%).

Use of lung SqCC specific markers, such as Desmoglein 3 and p40, may identify TTF-1 positive cases of squamous cell origin. Additionally, the use of Napsin A may confirm lung adenocarcinoma as the co-expression of Napsin A and TTF-1 in lung cancers has been shown to be more pulmonary specific than either one used alone (7). Finally, unlike clone 8G7G3/1, no cytoplasmic staining in lung cancers has been observed with clone SPT24 (8).

Principle of Procedure:

Antigen detection in tissues and cells is a multi-step immunohistochemical process. The initial step binds the primary antibody to its specific epitope. After labeling the antigen with a primary antibody, a one-, two- or three-step detection procedure can be employed. The one-step procedure will feature an enzyme-labeled polymer that binds to the primary antibody. A two-step procedure will feature a secondary antibody added to bind to the primary antibody. An enzyme-labeled polymer is then added to bind to the secondary antibody. The three-step detection procedure will feature a secondary

Principle of Procedure Cont'd:

antibody added to bind to the primary antibody followed by a linker antibody step for maximum binding. An enzyme-labeled polymer is then added to bind to the linker antibody. These detections of the bound antibodies are evidenced by a colorimetric reaction.

Source: Mouse monoclonal

Species Reactivity: Human; others not tested

Clone: SPT24

Isotype: IgG1/kappa

Protein Concentration: Call for lot specific Ig concentration.

Epitope/Antigen: TTF-1 (Thyroid transcription factor-1)

Cellular Localization: Nuclear

Positive Tissue Control: Lung adenocarcinoma

Negative Tissue Control: Normal liver, lung squamous cell carcinoma

Known Applications:

Immunohistochemistry (formalin-fixed paraffin-embedded tissues)

Supplied As: Buffer with protein carrier and preservative

Storage and Stability:

Store at 2°C to 8°C. The product is stable to the expiration date printed on the label, when stored under these conditions. Do not use after expiration date. Diluted reagents should be used promptly; any remaining reagent should be stored at 2°C to 8°C.

Protocol Recommendations (VALENT® Automated Slide Staining Platform):

VLTM3126 is intended for use with the VALENT. Refer to the User Manual for specific instructions for use. Protocol parameters in the Protocol Manager should be programmed as follows:

Deparaffinization: Deparaffinize for 8 minutes with Val DePar.

Pretreatment: Perform heat retrieval at 98°C for 60 minutes using Val AR-Hi pH, 5X (use at 1X).

Peroxidase Block: Block for 5 minutes with Val Peroxidase Block.

Protein Block (Optional): Incubate for 10-20 minutes with Val Background Block.

Primary Antibody: Incubate for 30 minutes.

Secondary: Incubate for 10 minutes with Val Mouse Secondary.

Linker: Incubate for 10 minutes with Val Universal Linker.

Polymer: Incubate for 10 minutes with Val Universal Polymer.

Chromogen: Incubate for 5 minutes with Val DAB.

Counterstain: Counterstain for 5 minutes with Val Hematoxylin.

Protocol Recommendations (intelliPATH FLX® and manual use):

Peroxide Block: Block for 5 minutes with Peroxidized 1.

Pretreatment: Perform heat retrieval using Diva Decloaker. Refer to the Diva Decloaker data sheet for specific instructions.

Protein Block (Optional): Incubate for 5-10 minutes at RT with Background Punisher.

Primary Antibody: Incubate for 30 minutes at RT.

Probe: Incubate for 10 minutes at RT with a secondary probe.

Polymer: Incubate for 10-20 minutes at RT with a tertiary polymer.

TTF-1 [SPT24]

Concentrated and Prediluted Monoclonal Antibody
901-3126-052623

BIOCARE
M E D I C A L

Protocol Recommendations (intelliPATH FLX and manual use)

Cont'd:

Chromogen: Incubate for 5 minutes at RT with Biocare's DAB – OR – Incubate for 5-7 minutes at RT with Warp Red.

Counterstain:

Counterstain with hematoxylin. Rinse with deionized water. Apply Tacha's Bluing Solution for 1 minute. Rinse with deionized water.

Technical Note:

This antibody, for intelliPATH FLX and manual use, has been standardized with MACH 4 detection system. Use TBS for washing steps.

Protocol Recommendations (ONCORE™ Automated Slide Staining System):

OAI3126 is intended for use with the ONCORE. Refer to the User Manual for specific instructions for use. Protocol parameters in the Protocol Editor should be programmed as follows:

Protocol Name: TTF-1 [SPT24]

Protocol Template (Description): Ms HRP Template 1

Dewaxing (DS Option): DS2

Antigen Retrieval (AR Option): AR1, high pH; 103°C

Reagent Name, Time, Temp.: TTF-1 [SPT24], 30 min., 25°C

Protocol Recommendations (ONCORE™ Pro Automated Slide Staining System):

OPAI3126 is intended for use with the ONCORE Pro. Refer to the User Manual for specific instructions for use. Protocol parameters in the Protocol Editor should be programmed as follows:

Protocol Name: TTF-1 [SPT24]

Protocol Template (Description): Ms HRP Template 1

Dewaxing (DS Buffer Option): DS2-50

Antigen Retrieval (AR Option): AR1, high pH; 103°C

Block Option: Buffer

Reagent Name, Time, Temp.: TTF-1 [SPT24], 30 min., 37°C

Limitations:

The optimum antibody dilution and protocols for a specific application can vary. These include, but are not limited to fixation, heat-retrieval method, incubation times, tissue section thickness and detection kit used. Due to the superior sensitivity of these unique reagents, the recommended incubation times and titers listed are not applicable to other detection systems, as results may vary. The data sheet recommendations and protocols are based on exclusive use of Biocare products. Ultimately, it is the responsibility of the investigator to determine optimal conditions.

Quality Control:

Refer to CLSI Quality Standards for Design and Implementation of Immunohistochemistry Assays; Approved Guideline-Second edition (I/LA28-A2) CLSI Wayne, PA USA (www.clsi.org). 2011

Precautions:

1. This antibody contains less than 0.1% sodium azide. Concentrations less than 0.1% are not reportable hazardous materials according to U.S. 29 CFR 1910.1200, OSHA Hazard communication and EC Directive 91/155/EC. Sodium azide (NaN₃) used as a preservative is toxic if ingested. Sodium azide may react with lead and copper plumbing to form highly explosive metal azides. Upon disposal, flush with large volumes of water to prevent azide build-up in plumbing. (Center for Disease Control, 1976, National Institute of Occupational Safety and Health, 1976) (9)

Precautions Cont'd:

2. Specimens, before and after fixation, and all materials exposed to them should be handled as if capable of transmitting infection and disposed of with proper precautions. Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents and specimens. If reagents or specimens come into contact with sensitive areas, wash with copious amounts of water. (10)
3. Microbial contamination of reagents may result in an increase in nonspecific staining.
4. Incubation times or temperatures other than those specified may give erroneous results. The user must validate any such change.
5. Do not use reagent after the expiration date printed on the vial.
6. The SDS is available upon request and is located at <http://biocare.net>.

Troubleshooting:

Follow the antibody specific protocol recommendations according to data sheet provided. If atypical results occur, contact Biocare's Technical Support at 1-800-542-2002.

References:

1. Di Loreto C, *et al.* Immunocytochemical expression of tissue specific transcription factor-1 in lung carcinoma. *J Clin Pathol.* 1997 Jan; 50(1):30-2.
2. Tacha D, *et al.* A 6-antibody panel for the classification of lung adenocarcinoma versus squamous cell carcinoma. *Appl Immunohistochem Mol Morphol.* 2012 May; 20 (3):201-7.
3. Brown AF, *et al.* Tissue-preserving antibody cocktails to differentiate primary squamous cell carcinoma, adenocarcinoma, and small cell carcinoma of lung. *Arch Pathol Lab Med.* 2013 Sep; 137(9):1274-81.
4. Masai K, *et al.* Expression of squamous cell carcinoma markers and adenocarcinoma markers in primary pulmonary neuroendocrine carcinomas. *Appl Immunohistochem Mol Morphol.* 2013 Jul; 21(4):292-7.
5. Matoso A, *et al.* Comparison of thyroid transcription factor-1 expression by 2 monoclonal antibodies in pulmonary and nonpulmonary primary tumors. *Appl Immunohistochem Mol Morphol.* 2010 Mar; 18(2):142-9.
6. Ordóñez NG. Value of thyroid transcription factor-1 immunostaining in tumor diagnosis: a review and update. *Appl Immunohistochem Mol Morphol.* 2012 Oct; 20 (5):429-44.
7. Bishop JA, Sharma R, Illei PB. Napsin A and thyroid transcription factor-1 expression in carcinomas of the lung, breast, pancreas, colon, kidney, thyroid, and malignant mesothelioma. *Hum Pathol.* 2010 Jan; 41(1):20-5.
8. Bejarano PA, Mousavi F. Incidence and significance of cytoplasmic thyroid transcription factor-1 immunoreactivity. *Arch Pathol Lab Med.* 2003 Feb; 127(2):193-5.
9. Center for Disease Control Manual. Guide: Safety Management, NO. CDC-22, Atlanta, GA. April 30, 1976 "Decontamination of Laboratory Sink Drains to Remove Azide Salts."
10. Clinical and Laboratory Standards Institute (CLSI). Protection of Laboratory Workers from Occupationally Acquired Infections; Approved Guideline-Fourth Edition CLSI document M29-A4 Wayne, PA 2014.