

## Dewax Solution Kit

Deparaffinization Solutions  
for the ONCORE Pro Automated Slide Stainer  
901-OPRI6004K-111620

**Catalog Number:** OPRI 6004K T60

**Description:** 60 tests

### Intended Use:

For In Vitro Diagnostic Use

Dewax Solution Kit consists of ready-to-use solutions for the removal of paraffin wax from formalin-fixed, paraffin-embedded (FFPE) tissues, as part of an immunohistochemistry (IHC) procedure on Biocare Medical's ONCORE Pro Automated Slide Stainer. The clinical interpretation of any staining or its absence should be complemented by morphological studies and proper controls and should be evaluated within the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

### Summary & Explanation:

Dewax Solutions 1 and 2 (DS1, DS2) are water-based solutions for the removal of paraffin wax from FFPE tissue specimens, as part of an IHC staining procedure on the ONCORE Pro Automated Slide Stainer. Removal of paraffin wax must be performed prior to antigen retrieval and the application of reagents for immunostaining. The solutions are provided ready-to-use and are intended to be applied sequentially as defined by the staining protocols on the ONCORE Pro Automated Slide Stainer.

### Known Applications:

Immunohistochemistry (FFPE tissues)

### Reagents Provided:

Dewax Solution Kit is comprised of ready-to-use solutions. Sufficient reagent is provided to perform a total of 60 tests:

Dewax Solution 1, DS1 (OPRI6001 T30) 60 tests (10.5 mL x2)

Dewax Solution 2, DS2 (OPRI6003 T30) 60 tests (9.0 mL x2)

### Reconstitution, Dilution and Mixing:

Dewax Solutions are provided ready-to-use. No reconstitution, dilution or mixing is required.

### Materials and Reagents Required but Not Provided:

Reagents and materials, such as primary antibodies, detection kits, chromogens and ancillary reagents are not provided. Refer to the ONCORE Pro Automated Slide Staining System User Manual for a complete list of materials and reagents required.

### Storage and Stability:

Store at 2°C to 8°C. The product is stable to the expiration date printed on the label, when stored under these conditions. Do not use after expiration date.

### Instructions for Use:

Note: DS2 is referred as DS2-50 on ONCORE Pro Automated Slide Stainer. Use DS2 when DS2-50 is required.

Dewax Solutions 1 and 2 are provided in vials ready for use on the ONCORE Pro Automated Slide Stainer. Uncap the vial and place in the ONCORE Pro reagent tray. The ONCORE Pro Automated Slide Stainer will apply each dewax reagent as required in the selected protocol.

### Instructions for Use Cont'd:

Refer to the appropriate antibody data sheet for the recommended staining protocol. Some antibodies may require use of DS Buffer in place of DS2. Therefore, it is expected to observe significant quantities of DS2 remaining in the reagent vials after 60 tests as only one of these two reagents is consumed in each test. DS Buffer is applied directly from the Buffer Carboy. Refer to the ONCORE Pro Automated Slide Staining System User Manual for detailed instructions on instrument operation and additional protocol options.

### Limitations:

These reagents have been optimized for use with ONCORE Pro antibodies and ancillary reagents. The protocols for a specific application can vary. These include, but are not limited to fixation, heat-retrieval method, incubation times, and tissue section thickness. Third party primary antibodies may be used on the ONCORE Pro; however, appropriate antibody concentration may depend upon multiple factors and must be empirically determined by the user. Ultimately, it is the responsibility of the investigator to determine optimal conditions.

### Quality Control:

Refer to CLSI Quality Standards for Design and Implementation of Immunohistochemistry Assays; Approved Guideline-Second edition (I/LA28-A2) CLSI Wayne, PA USA (www.clsi.org). 2011

### Precautions:

1. Refer to reagent Safety Data Sheet for precautions.
2. Specimens, before and after fixation, and all materials exposed to them should be handled as if capable of transmitting infection and disposed of with proper precautions. Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents and specimens. If reagents or specimens come in contact with sensitive areas, wash with copious amounts of water. (3)
3. Microbial contamination of reagents may result in an increase in nonspecific staining.
4. Incubation times or temperatures other than those specified may give erroneous results. The user must validate any such change.
5. Do not use reagent after the expiration date printed on the vial.



**FLAMMABLE**



**IRRITANT**



**ENVIRONMENTAL HAZARD**

### Troubleshooting:

Follow the antibody specific protocol recommendations according to data sheet provided. If atypical results occur, contact Biocare's Technical Support at 1-800-542-2002.

## Dewax Solution Kit

Deparaffinization Solutions  
for the ONCORE Pro Automated Slide Stainer  
901-OPRI6004K-111620

### References:

1. Taylor CR, Cote RJ. Immunomicroscopy: A Diagnostic Tool for the Surgical Pathologist. 3rd Ed. Philadelphia: Saunders Elsevier, 2006.
2. Dabbs DJ. Diagnostic Immunohistochemistry: Theranostic and Genomic Applications. 3rd Ed. Philadelphia: Saunders Elsevier, 2010.
3. Clinical and Laboratory Standards Institute (CLSI). Protection of Laboratory Workers from Occupationally Acquired Infections; Approved Guideline-Fourth Edition (M29-A4) Wayne, PA 2014.