

Use of a Novel Rabbit Monoclonal Phospho-Histone H3 (Ser10) versus H&E Mitotic Count in Melanoma

Regan Fulton, MD, PhD and David Tacha, PhD, HTL (ASCP)

Introduction

Mitotic count is a known prognostic indicator in cutaneous melanoma.¹ The traditional hematoxylin and eosin (H&E) manner of counting mitotic figures (MFs) using a 40X objective is time consuming and prone to inter- and intra-observer variability.^{2,3} MFs can also be difficult to assess in H&E-stained sections when poor fixation is observed.⁴ Ser10 on histone H3 is phosphorylated in association with mitotic chromatin condensation in late G2 and M phase of the cell cycle and thus phospho-histone H3 (pHH3) can distinguish mitotic from apoptotic nuclei.⁵ To date, a pHH3 polyclonal antibody has seemingly been a reliable marker for mitotic counting in different types of tumors.^{1-4,6-8} Recently, a new commercially available rabbit monoclonal mitotic marker Ser10 pHH3 has been developed and was shown to be more specific than the widely used polyclonal pHH3.⁵ In certain melanoma cases, granulated nuclei at interphase were much more prominent with polyclonal pHH3 compared to the rabbit monoclonal pHH3, making it more difficult to interpret.⁵ Rabbit polyclonal antibodies have been shown to have potential staining inconsistencies due to batch-to-batch variation and in most cases, monoclonal antibodies are generally preferred.⁵ This led us to investigate the use of the rabbit monoclonal pHH3 to count MFs and to compare counts with adjacent H&E sections.

Methods and Methods

Eight cases of whole tissue formalin-fixed paraffin embedded tissues were selected. Of the 8 cases, 2 cases showed poor fixation and 6 cases showed good fixation. Adjacent sections for each case were cut at 4 microns and stained with H&E. Rabbit monoclonal pHH3 [BC39] (Biocare Medical, Concord, CA) was applied to melanoma tissue sections, followed by a polymer detection system, 3,3'-diaminobenzidine for visualization and hematoxylin counterstain. Whole slides were examined at 20X with 40X confirmation of equivocal structures. The entirety of each tumor was examined and strict criteria were used for counting mitosis: only tumor cells showing chromatin condensation and finger-like projections of chromatin, were counted as mitosis (stroma and inflammatory infiltrate excluded). Apoptotic showing breakdown of karyorrhexis were excluded.

Discussion

The 6 cases with good histology/fixation had a total of 1823 mitotic counts by H&E vs. 3703 with pHH3, thus a 203% increase in MF (Figure 2 A, B). Other studies have also demonstrated doubling of mitotic counts using polyclonal pHH3, compared to standard H&E; in line with our observations.⁶⁻⁸ In the 2 cases with poor histology/fixation, 50 mitotic figures were observed by H&E vs. 265 with pHH3, representing a 530% increase in MF (Figure 3A, B). Mitotic cell counting with pHH3 was much more rapid than H&E mitotic cell counting (approximately 50% faster). We hypothesize that the overall discordance between pHH3 MFs compared to the H&Es can be explained by the high contrast and sensitivity of immunohistochemical staining for MF. Apoptotic cells and pyknotic nuclei remained unstained with pHH3, however it was noted that a few mitotic nuclei remain unstained by pHH3.

Figure 1

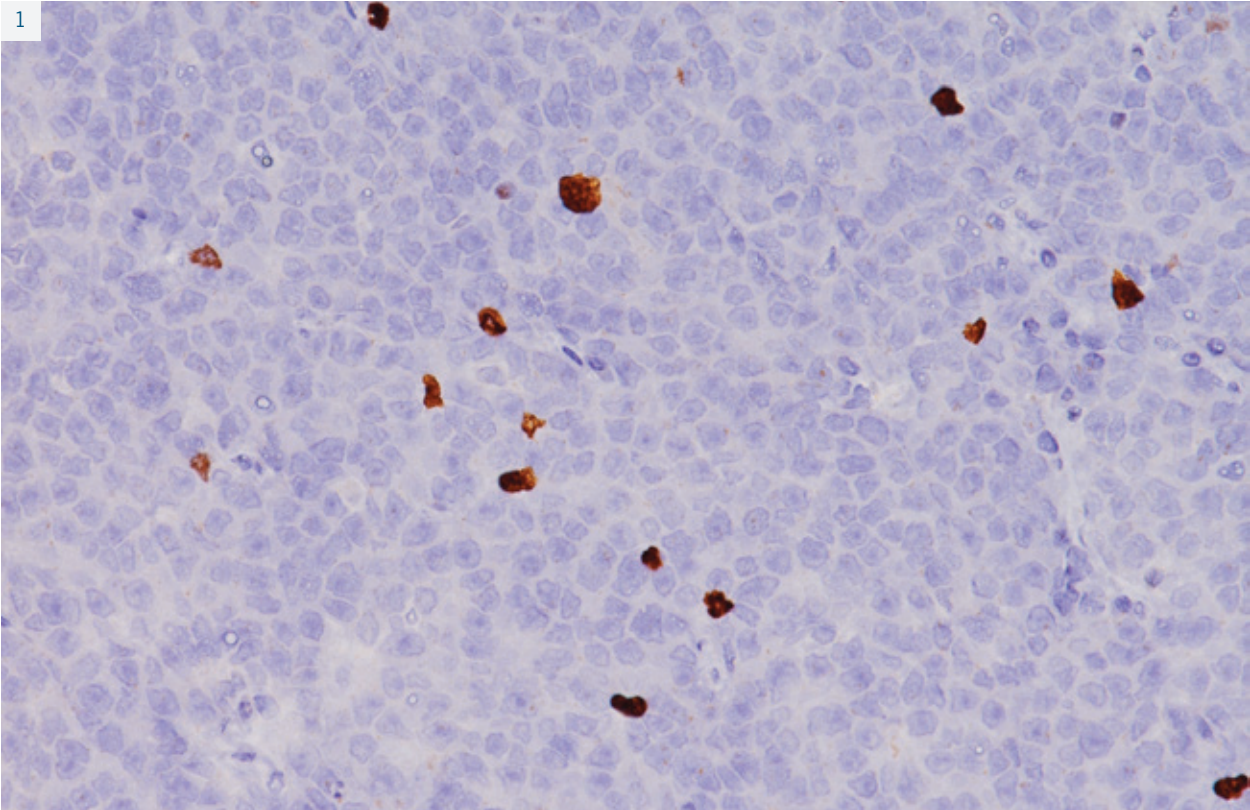


Figure 1: Rabbit monoclonal pHH3 in melanoma

Figure 2

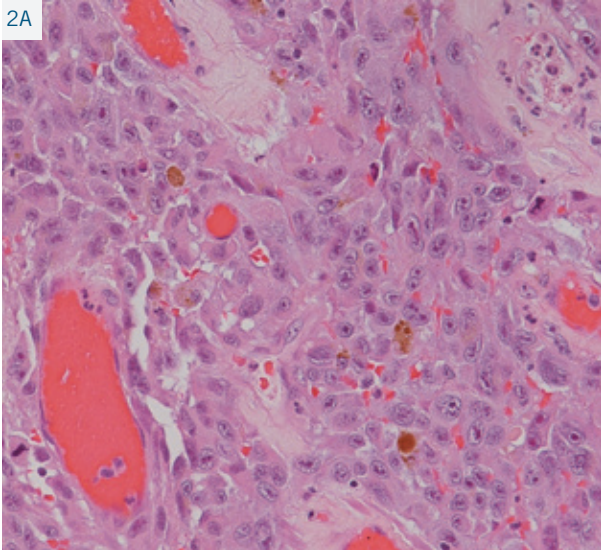


Figure 2A: Melanoma, H&E Good Fixation

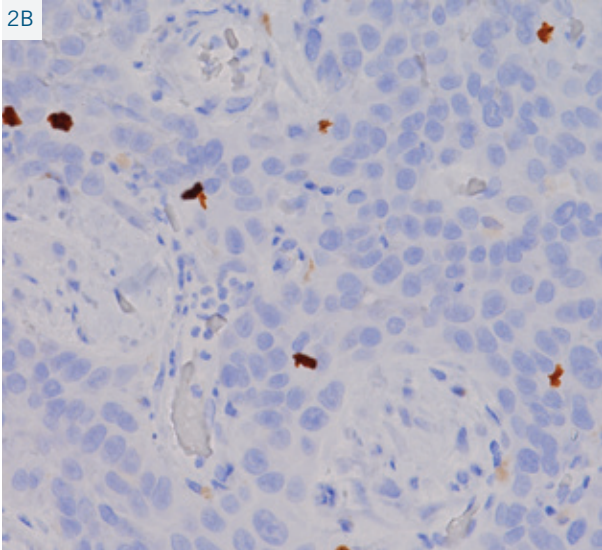


Figure 2B: Melanoma, pHH3 Good Fixation

Figure 3

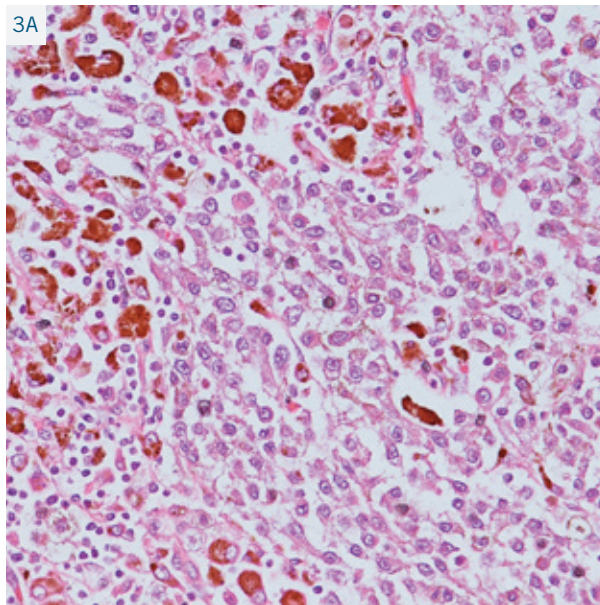


Figure 3A: Melanoma, H&E Poor Fixation

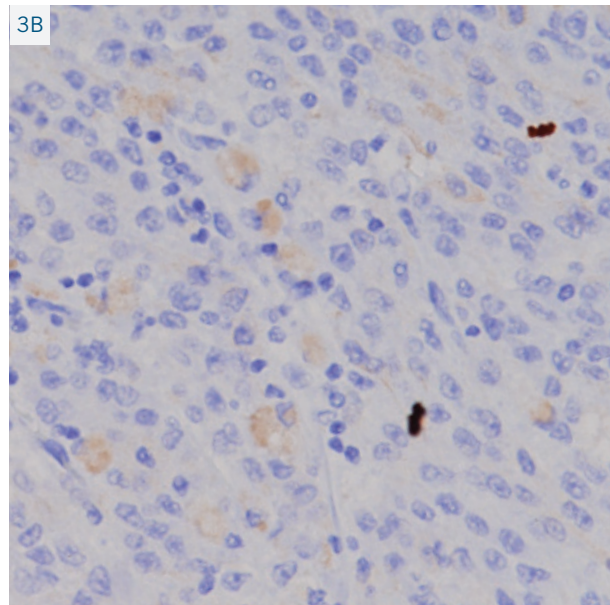


Figure 3B: Melanoma, pHH3 Good Fixation

Table 1

| Case | Histology Quality | Mitotic Count | pHH3 |
|------|-------------------|---------------|------|
| 1 | Good | 104 | 214 |
| 2 | Good | 250 | 775 |
| 3 | Good | 972 | 1343 |
| 4 | Good | 122 | 512 |
| 5 | Poor fixation | 24 | 180 |
| 6 | Good | 118 | 515 |
| 7 | Poor fixation | 26 | 185 |
| 8 | Good | 257 | 344 |

The pHH3 antibody produced high contrast staining for MFs was observed in all 8 cases of melanoma. Manual counting MFs in tumor cells were easily calculated with a light hematoxylin stain. (Figure 1)

Conclusion

In this study, we demonstrated clear advantages of using pHH3 for obtaining mitotic count over the traditional H&E method. MFs were more easily identified following immunohistochemical staining with pHH3; and it appears to be more accurate and reproducible, especially in cases of poorly-fixed tissues. Moreover, because MFs are more easily distinguished with pHH3 staining, mitotic count analysis is significantly faster when using pHH3 over traditional H&E.

References

1. Ladstein RG, *et al.* Prognostic importance of the mitotic marker phosphohistone H3 in cutaneous nodular melanoma. *J Invest Dermatol.* 2012 Apr;132(4):147-52.
2. Warth A, *et al.* Interobserver agreement of proliferation index (Ki-67) outperforms mitotic count in pulmonary carcinoids. *Virchows Arch.* 2013 May;462(5):507-13.
3. Casper DJ, *et al.* Use of anti-phosphohistone H3 immunohistochemistry to determine mitotic rate in thin melanoma.
4. Rakha EA, *et al.* Breast cancer prognostic classification in the molecular era: the role of histological grade. *Breast Cancer Res.* 2010;12(4):207.
5. Qi W, Tacha DE, Zhou D. A New Rabbit Monoclonal Phosphorylated-Histone H3 (pHH3) Hybridoma: An Immunohistochemical Comparison Study with a Rabbit Polyclonal pHH3; *Modern Pathology*, Poster Session, March 2015.
6. Ikenberg K, Pfaltz M, Rakozy C, Immunohistochemical dual staining as an adjunct in assessment of mitotic activity in melanoma. *J Cutan Pathol.* 2012 Mar;39(3):324-30.
7. Casper DJ, *et al.* Use of anti-phosphohistone H3 immunohistochemistry to determine mitotic rate in thin melanoma. *Am J Dermatopathol.* 2010 Oct;32(7):650-4.
8. Angi M, *et al.* Immunohistochemical assessment of mitotic count in uveal melanoma. *Acta Ophthalmol.* 2011 Mar;89(2):e155-60.