

# Biocare's UltraLine

Ready to Use Antibodies for Roche Instruments\*

# Biocare's UltraLine - Ready to Use Antibodies for Roche Instruments

While every vendor strives to provide their customers with a product catalog that is as comprehensive as possible, the nature of IHC makes it impossible for vendors to offer every marker in every format.

To address these needs, instrument vendors make it possible for customers to purchase empty reagent containers and use them to program and load custom third-party reagents onto their instruments.


However, although another vendor may offer a different format, a higher quality clone, or even the same clone at a lower price, labs may be reluctant to spend the time and resources required for optimizing an antibody to an instrument and detection kit that it was not originally developed for.

To overcome this obstacle, Biocare has released antibodies that have been specially developed for these detection kits. For laboratories that run their IHC stains on the Roche BenchMark ULTRA, Biocare has developed the Ultraline, a specialized line of antibodies that have been optimized for use with Roche ultraView and OptiView Detection.

Ultraline antibodies are supplied as 6mL ready-to-use predilutes designed to be transferred to a BenchMark ULTRA user-fillable dispenser prep kit. The recommended protocol and detection kit for each marker is included in the product datasheet, which reduces the burden of optimization on the customer. Each 6mL volume corresponds to 50 tests when loaded into the prep kit.

Biocare is constantly developing and releasing new Ultraline antibodies to our antibody catalog. For a list of all current Ultraline antibody offerings, please visit our website. A brochure with all current Ultraline markers can be found at <https://biocare.net/media/brochures-flyers/>.

## How to Use Biocare's UltraLine on Roche

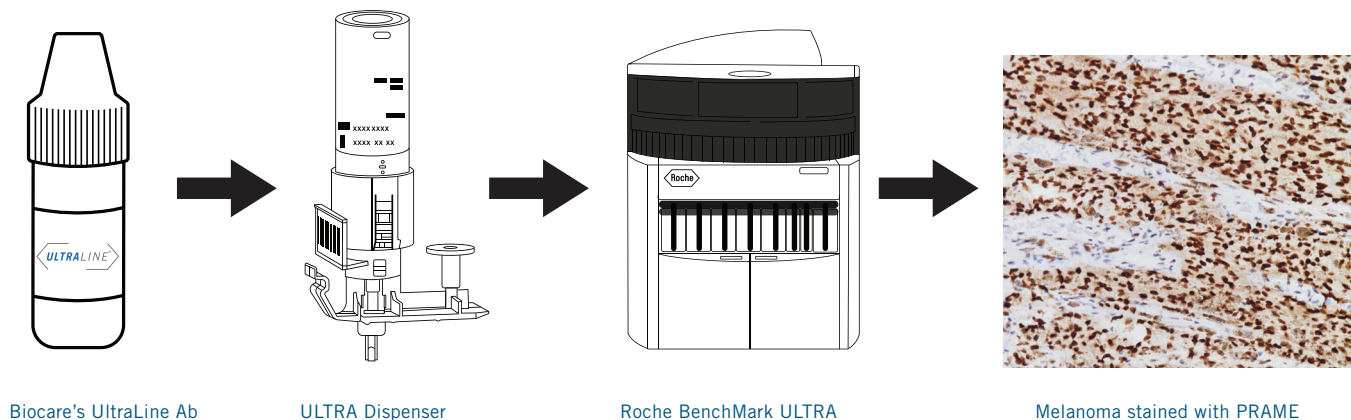

If there is a particular marker that your lab is interested in that is not currently available in Ultraline format, product requests can be made through your Biocare account executive.

**Simply contact your Biocare rep about your request and include the following information:**

- What is your lab currently using this marker for?
- How many slides is your lab running with this marker, either daily, weekly, monthly, or yearly?
- What instrument is your lab currently running this marker on?

The Biocare Ultraline is available for online purchase at <https://biocare.net/ultraline/>

Customers can also reach us by phone at 1-800-799-9499.

For any assistance on programming and running Ultraline antibodies, select option #3 for Technical Support.